


## FreeO<sub>2</sub> Setup

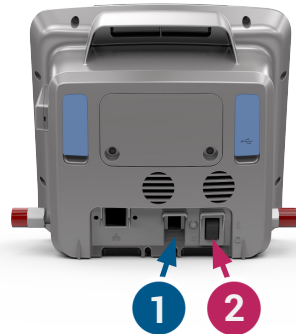

1. Plug in the power cord to the FreeO<sub>2</sub> **1** and connect the other end to the power outlet

2. Press the switch **2** in order to start the FreeO<sub>2</sub>



3. Connect the oxygen hose to the wall socket **3** (or the high pressure socket located at the top of the oxygen cylinder)

4. Lift the blue cover **4** and connect the Nonin brand SpO<sub>2</sub> sensor




5. Connect the patient interface **5** (standard mask or nasal cannula). Do not use high concentration mask.

*NB: It is necessary to connect an adaptor (see beside) between the oxygen output and the patient interface*



# FreeO<sub>2</sub> controls

1. select the option «New patient» to start a new treatment.

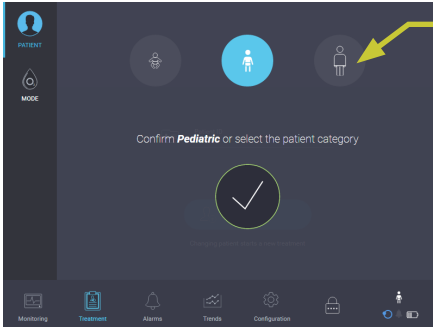

2. Select the type of patient in the upper part of the screen. From left to right:

- Infant - maximum O<sub>2</sub> flow rate limited to 5L/min
- Child - maximum O<sub>2</sub> flow rate limited to 10L/min
- Adult - maximum O<sub>2</sub> flow rate limited to 20L/min

Confirm your choice by pressing the check sign ✓




3. Select the FreeO<sub>2</sub> mode (automated Oxygen Therapy)

4. Adjust the SpO<sub>2</sub> Target using the + or - sign

5. (Optional) adjust a different maximum O<sub>2</sub> flow rate if needed\*

6. Press the «monitoring» icon to access the monitoring screen

## Patient monitoring

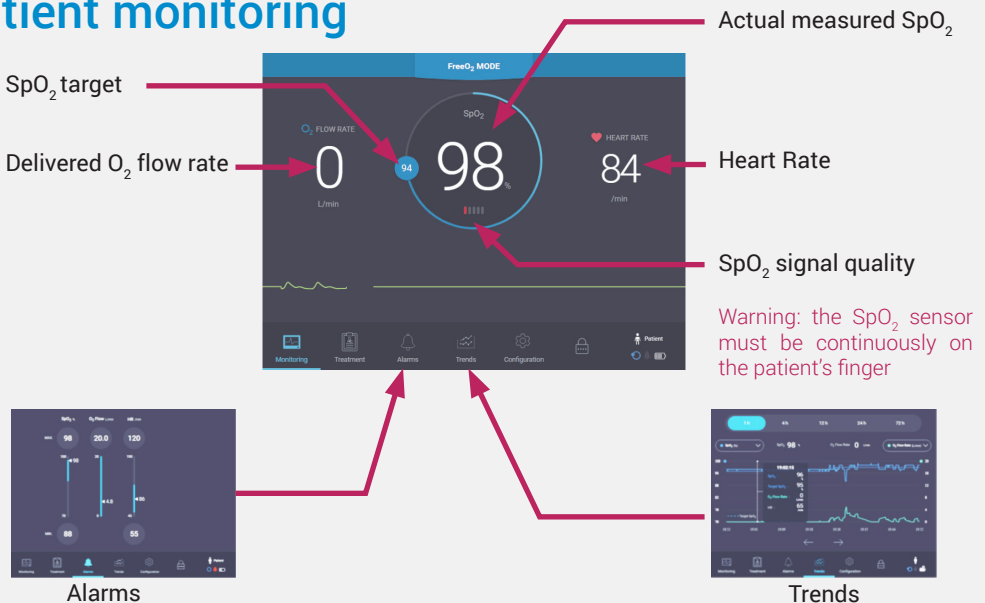

\* This manual limitation of the maximum delivered O<sub>2</sub> flowrate might limit the algorithm capacities to quickly adapt to the patient's needs in O<sub>2</sub>